

## **GB** COMPLETE RevitaLens Multi-Purpose Disinfecting Solution

**Important – Please read carefully and keep this package insert for future reference.**

COMPLETE RevitaLens Multi-Purpose Disinfecting Solution cleans, rinses, disinfects, stores, removes surface deposits including lipids and proteins, and re-wets soft (hydrophilic) contact lenses.

**Caring for your contact lenses with COMPLETE RevitaLens Multi-Purpose Disinfecting Solution.**

**Always wash and dry your hands thoroughly before handling your soft contact lenses.**

**Cleaning:**

- Place a few drops of COMPLETE RevitaLens Multi-Purpose Disinfecting Solution on each lens surface.
- Rub each side gently from the center to the edge for 2-4 seconds. Never rub in a circular motion, it may tear the lens and not clean the outer edge surface.

**Rinsing:**

- Rinse each side of the lens for at least 5 seconds with COMPLETE RevitaLens Multi-Purpose Disinfecting Solution.
- Place lenses in the correct chambers of the AMO lens case (blue for left - white for right).
- Fill both chambers with fresh COMPLETE RevitaLens Multi-Purpose Disinfecting Solution. Replace lens case caps and soak for at least 6 hours for disinfection, protein and lipid removal.

Your lenses are now ready for wear. Rinse the lens case with COMPLETE RevitaLens Multi-Purpose Disinfecting Solution and leave to air dry.

**Storage:** You may leave your lenses in the unopened lens case in COMPLETE RevitaLens Multi-Purpose Disinfecting Solution for up to 30 days continuously. After 30 days, cleaning and disinfection must be repeated before wearing your lenses.

**Intensive Cleaning:** COMPLETE RevitaLens Multi-Purpose Disinfecting Solution removes protein from your lenses during cleaning and disinfection. This will help to keep your lenses smooth for enhanced wearing comfort. Where the level of protein deposits is unusually high a more intensive protein removal may be necessary. Your lens care practitioner will advise if you require protein removal tablets.

**WARNINGS:**

**All instructions must be followed. Failure to do so may result in serious eye infection or injury. If eye irritation occurs immediately discontinue lens wear and contact your lens care practitioner.**

**Precautions:**

**General:** Use COMPLETE RevitaLens Multi-Purpose Disinfecting Solution with soft contact lenses, including silicone hydrogels lenses  
Do not use if allergic to any of the ingredients  
Do not wear your lenses longer than prescribed  
Discard any lenses that are damaged or dehydrated  
Avoid using aerosols when wearing contact lenses  
Attend regular check-ups with your lens care practitioner  
Keep out of the reach of children

**Use:** Only use COMPLETE RevitaLens Multi-Purpose Disinfecting Solution with the lens case provided  
Do not use if tamper evident seal is missing or broken  
Always use fresh solution  
Always manually rub your lenses with solution for at least 2-4 seconds before soaking  
Do not swim, shower or bath while wearing contact lenses  
Do not use saliva or put lenses in your mouth to rewet  
Do not use after the expiry date  
To avoid contamination do not touch the dropper tip to any surface  
Do not swallow the solution  
Do not rinse and store your lenses or lens case with tap water

**Storage:** Close the bottle tightly immediately after use  
Store at room temperature

**Discard and Expiry:** Use before the expiry date on label and carton  
Discard any remaining solution 90 days after first opening the bottle  
Discard any remaining solution in your contact lens case before disinfecting your lenses again  
Always discard your old lens case and use the new one provided

**Product:** COMPLETE RevitaLens Multi-Purpose Disinfecting Solution

**Ingredients:** Alexidine 0.00016%, Polyquaternium-1 0.0003%, Boric acid, Sodium borate decahydrate, Tetronic 904, Edetate disodium (EDTA), Trisodium citrate dihydrate, Sodium chloride and Purified water.

This product does not contain phthalates. This product does not contain BPA.