

Ultrazyme Universal Intensive Protein Cleaner

INTENSIVE CLEANING OF YOUR CONTACT LENSES

Ultrazyme Universal Intensive Protein Cleaner removes the excess protein which normal cleaning leaves behind. Regular use makes lenses smooth and comfortable for maximum lens life and clarity of vision.

For maximum benefit use Ultrazyme Universal Intensive Protein Cleaner tablets once a week or as directed by your eye care professional

Oxysept 1 Step System

How to use Ultrazyme Universal Intensive Protein Cleaner with Oxysept 1 Step Disinfecting and Neutralising System.



- Once a week, add one Ultrazyme Universal Intensive Protein Cleaner tablet to the Oxysept 1 Step disinfecting solution in addition to the neutralising tablet. Follow your normal disinfecting and neutralising routine.



- Leave for a minimum of 6 hours but do not leave your lenses for longer than 12 hours in the solution to which the Ultrazyme Universal Intensive Protein Cleaner tablet has been added.



- After using Ultrazyme Universal Intensive Protein Cleaner tablets always rub and rinse both sides of your lenses with sterile saline solution before inserting them into your eyes.

Rinse your lens case with a sterile saline solution and allow to air dry

TotalCare Disinfecting Solution

How to use Ultrazyme Universal Intensive Protein Cleaner with TotalCare Disinfecting Solution.



- Once a week, add one Ultrazyme Universal Intensive Protein Cleaner tablet to the lens case filled with TotalCare Solution and close it.



- Allow your lenses to soak for at least 6 hours but no longer than 12 hours in the solution containing the Ultrazyme Universal Intensive Protein Cleaner tablet.



- After using Ultrazyme Universal Intensive Protein Cleaner tablets, always rub and rinse your lenses thoroughly with TotalCare Solution before putting them into your eyes.

Rinse the lens case with TotalCare Solution and allow it to air dry.

Ultrazyme Universal Intensive Protein Cleaner

PRECAUTIONS

All instructions must be followed. Failure to do so may result in serious eye infection or injury. Always wash and dry your hands thoroughly before handling your contact lenses.

GENERAL

- Do not use any of these products if you are allergic to any of the ingredients.
- If eye irritation occurs discontinue lens wear and consult your contact lens practitioner.
- Only use with the appropriate AMO solutions and single chamber lens case.

USE

- Do not insert lens directly into the eye from the Ultrazyme Universal Intensive Protein Cleaner solution or put the solution containing the tablet directly into your eye. Should this happen, remove the lens immediately and rinse the eye with water.
- Do not put Ultrazyme Universal Intensive Protein Cleaner tablets in a double chambered lens case.
- Do not use tablets that are soft, sticky or irregular in appearance.
- Do not swallow these tablets.
- Do not use after the expiry date.
- Do not rinse your lenses or lens case with tap water.
- Do not re-use the solution.
- After use, always empty the lens case, rinse with TotalCare Disinfecting Solution or sterile saline solution and allow to air dry.

STORAGE

- Keep out of reach of children.
- Store in a dry place at room temperature.

CONTAINS:

Subtilisin A (0.4mg/tablet)
Acetyl Cysteine
Polyethylene Glycol 3350
Sodium Carbonate, Anhydrous
Sorbitol
1-Tartaric Acid

Symbols



CE 0483

AMO Ireland
Block B, Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

© 2010 Abbott Medical Optics Inc.

 **Abbott**

AMO, Oxysept, Oxysept 1 Step and Ultrazyme are trademarks owned by or licensed to Abbott Laboratories, its subsidiaries or affiliates.

© 2010 Abbott Medical Optics Inc.

AM70318GB10H
Revision Date: 11/2010

 **Abbott**